uNitas™

anterior cervical plate system

+ Maximale Flexibilität
+ Maximum flexibility

100 years

Ulrich
medical

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100 years of reliability and German precision

Spinal systems by ulrich medical® stand for quality “Made in Germany.”

They are the successful result of systematic development activities and many years of experience in medical technology. For more than 100 years, we have done our best every day for our customers and for patient health with our proprietary and innovative products.

Leading-edge technology and competence from one source

		Seite Page	
	Einleitung	4	1
	Introduction	4	
	Indikationen und Kontraindikationen	5	2
	Indications and contraindications	5	
	uNitas™ Implantate	6 8	3
	uNitas™ implants	6 8	
	uNitas™ Instrumente	9 11	4
	uNitas™ instruments	9 11	
	Operationstechnik	12 24	5
	Surgical technique	12 24	
	Sieb	25	6
	Tray	25	
	Komponenten	26 29	7
	Components	26 29	

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The small pictures at the bottom of the page show the step-by-step application of the instruments that are used as per the surgical steps on the double page. Pictures with instruments that had been used before are blue-colored.

uNitas™ is a cervical plate system for the anterior surgical stabilization of the cervical spine, with the exception of C0-C2. The implant enables the immobilization of the fusion mass until consolidation of the implant or bone graft. It consists of single- or multisegmental plates as well as expandable screws and small fragment bone screws of varying sizes.

The special feature of the system is its outstanding flexibility. The instrumentation can be adjusted depending in the situation through the elective use of bicortical bone screws and self-drilling, monocortical bone screws. Bone screws with a larger diameter are also available, as are expandable screws which enable optimized stabilization and maximum hold during revision procedures or in the case of reduced bone quality.

uNitas™ implant components are made according to the DIN EN ISO 9001 standard from a titanium alloy ISO 5832-3 (TiAl6V4). The material is biocompatible, corrosion-resistant, non-toxic under biological conditions and does not interfere with imaging procedures such as X-ray imaging, computer tomography and MRIs (Magnetic Resonance Imaging).

The present description of the surgical technique contains the work steps for the use of the uNitas™ system. The implants and the system instruments needed for implantation are presented.

The use of this manual is not sufficient as sole basis for the successful application of the uNitas™ system. It is recommended to master the surgical technique with an experienced surgeon.

Please refer to the instructions for use and the instrument processing instructions.

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Indications

uNitas™ is used in case of instabilities of different genesis such as

- states after removal of the disk
- fracture
- tumor
- pseudarthroses
- after unsuccessful cervical spine surgeries

Contraindications

- Patients with acute infection, whether superficial or deep
- Patients with a history of material allergy or who tend to react to foreign bodies
- The physician must consider carefully before treating patients who are in a generally unfavorable medical or psychological state and who could be made worse by the procedure
- Pregnancy

Please note the instructions for use for uNitas™ implants and the instrument processing instructions.

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Corpectomy C6, interbody fusion with iliac crest and anterior plating with uNitas™

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In the uNitas™ system are four types of screws for monocortical and bicortical use available:

Artikelnummer Product number	Beschreibung Description	Länge Length	Ø Ø	Farbe Color
CS 4200-40-12	Knochenschraube, selbstschneidend Bone screw, self tapping	12 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-14	Knochenschraube, selbstschneidend Bone screw, self tapping	14 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-16	Knochenschraube, selbstschneidend Bone screw, self tapping	16 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-18	Knochenschraube, selbstschneidend Bone screw, self tapping	18 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-20	Knochenschraube, selbstschneidend Bone screw, self tapping	20 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-22	Knochenschraube, selbstschneidend Bone screw, self tapping	22 mm	4,0 mm 4.0 mm	gelb yellow
CS 4200-40-24	Knochenschraube, selbstschneidend Bone screw, self tapping	24 mm	4,0 mm 4.0 mm	gelb yellow
CS 4201-40-12	Knochenschraube, selbstbohrend Bone screw, self drilling	12 mm	4,0 mm 4.0 mm	grün green
CS 4201-40-14	Knochenschraube, selbstbohrend Bone screw, self drilling	14 mm	4,0 mm 4.0 mm	grün green
CS 4201-40-16	Knochenschraube, selbstbohrend Bone screw, self drilling	16 mm	4,0 mm 4.0 mm	grün green
CS 4201-40-18	Knochenschraube, selbstbohrend Bone screw, self drilling	18 mm	4,0 mm 4.0 mm	grün green
CS 4202-40-12	Spreizschraube inkl. Stift, expandierbar expandable screw incl. bolt	12 mm	4,0 mm 4.0 mm	blau blue
CS 4202-40-14	Spreizschraube inkl. Stift, expandierbar expandable screw incl. bolt	14 mm	4,0 mm 4.0 mm	blau blue
CS 4202-40-16	Spreizschraube inkl. Stift, expandierbar expandable screw incl. bolt	16 mm	4,0 mm 4.0 mm	blau blue
CS 4202-40-18	Spreizschraube inkl. Stift, expandierbar expandable screw incl. bolt	18 mm	4,0 mm 4.0 mm	blau blue
CS 4203-45-12	Knochenschraube, selbstschneidend Bone screw, self tapping	12 mm	4,5 mm 4.5 mm	grau grey
CS 4203-45-14	Knochenschraube, selbstschneidend Bone screw, self tapping	14 mm	4,5 mm 4.5 mm	grau grey
CS 4203-45-16	Knochenschraube, selbstschneidend Bone screw, self tapping	16 mm	4,5 mm 4.5 mm	grau grey
CS 4203-45-18	Knochenschraube, selbstschneidend Bone screw, self tapping	18 mm	4,5 mm 4.5 mm	grau grey

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Determining plate length

The preoperative determination of the suitable plate length is performed using the available patient data (X-rays or CT/MRI images).


The intraoperative determination of the suitable plate length is performed using the plate measuring instrument (CS 4229).


After the plate length has been measured in-situ, the corresponding plate length can be easily read by placing the plate measuring instrument (CS 4229) on the scale of the layer 1 for uNitas™ plates and screws (CS 4232-01).

The measured plate length (mm) refers to the total length of the plate, not to the distance between the holes.

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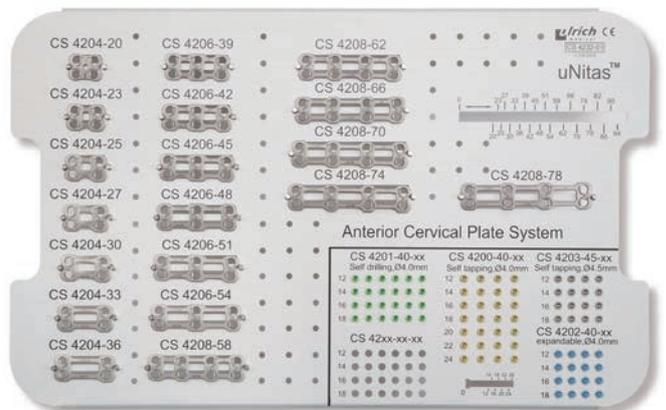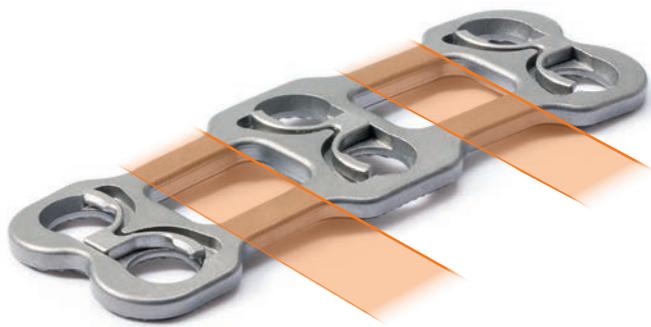(Re-) Bending the plate

The uNitas™ plate is lordotically pre-bent. If rebending of the plate is necessary, this can be done using the plate bender (CS 4223). The implant should only be bent in the area of the plate-bending zones.

Important: Only the intended instrument, the plate bender (CS 4223), should be used for bending the plate. Repeated bending back and forth should be avoided. This may lead to damage or breakage of the implant.

The plate should not (re-) bent in the area of the back-out mechanism.

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Placement of the plate

The selected plate is brought into the desired position using the plate holder (CS 4224).

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Optional: Temporary plate fixation

After checking the correct plate length and the correct positioning, a temporary fixation of the plate to the vertebral body can be achieved using the inserter for pin (CS 4221) by inserting the fixation pin (CS 4220).

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To insert the fixation pin (CS 4220), push back the sleeve of the inserter for pin (CS 4221). The pin can now be inserted. When the sleeve is released, the pin is firmly locked.

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The fixation pins are reusable. Discard any pins that are bent.

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Optional: Opening the cortical bone using an awl

Using the awl (CS 4216), the cortical bone can be opened by the single (CS 4225) or double drill guide (CS 4226).

The AO handle (CS4215) is mounted on the awl (CS 4216) for this purpose.

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Use of the depth adjuster

The depth adjuster (CS 4227) is pushed onto the awl (CS 4216). To do this, the internal locking mechanism is released by pressing together the depth adjuster (CS 4227).

The depth adjuster (CS 4227) can now be pushed onto the awl (CS 4216). After releasing the compression, the depth adjuster is fixed in the desired position.

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The set length can be read on the gauge.

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Optional: Opening the cortical bone using an awl

The cortical bone can now be opened using the awl (CS 4216) and the single (CS 4225) or double drill guide (CS 4226).

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Drilling the screw holes:

Self-tapping bone screws (CS 4200-40-xx) are available in the uNitas™ system for bicortical fixation.

Use the drill (CS 4218) to prepare the screw holes. Depending on preference, the AO-handle (CS 4215) can be mounted on the drill and manually drilled.

Knochenschraube, selbstschneidend
CS 4200-40-xx
Bone screw, self tapping
CS 4200-40-xx

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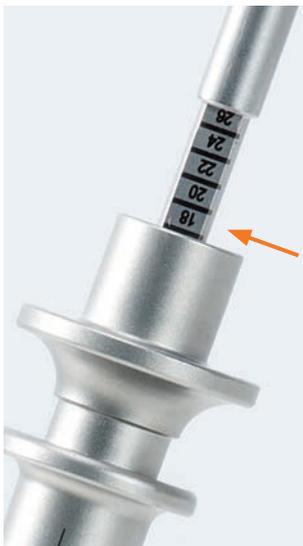
The depth adjuster (CS 4227) is used analogously to the awl (CS 4216). See page 15.

The length for the drill or the awl is determined using patient imaging data (X-rays or CT/MRI images).

The screw length corresponds to the drilling depth determined by the depth adjuster.

Important: Check that the depth adjuster is properly locked in order to avoid drilling too deeply. Perform drilling using the drill guide and C-arm control!

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Use of the drill guides:

The single (CS 4225) or double drill guide (CS 4226) is used to drill the screw holes. The drill (CS 4218) Ø 2.0 mm is introduced through the drill guide.

The single drill guide (CS 4225) can be used at an angle of +/- 7.5°.

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Alternatively, the double drill guide (CS 4226) can be used. The double drill guide is inserted in the hole pair of a plate level. The angulation is defined at 6° on a medial plane. On a sagittal plane, the angulation can be varied by +/- 7.5°.

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Important: It should be ensured that the angulation of +/- 7.5° is not exceeded. A larger angulation can cause the screw to incorrectly secure the plate on the bone and/or the locking mechanism of the screw to not be fully snapped into place.

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Optional: Taps

All uNitas™ screws are self-tapping. If required by the situation, the screw hole can be correspondingly prepared using the tap (CS 4217) Ø 4.0 mm.

The depth adjuster (CS 4227) is used analogously to the awl (CS 4216). See page 15.

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Screw implantation:

Use of the bone screw, bicortical

The screw holder (CS 4222) is used for screw implantation and to help remove the desired screw directly from the tray insert. To insert the screw, the sleeve of the screw holder is pushed back. The gripper arms open and the screw can be grasped. When the sleeve is released, the screw is firmly attached to the screw holder.

Please refer to the assembly/disassembly instructions (WS 4285-01) for CS 4222.

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Use of the screw holder

The screwdriver (CS 4219) is now introduced into the screw holder (CS 4222). A lock prevents the screw holder from being able to slip off the screwdriver.

The screws are implanted as close to the cover plate as possible using the screwdriver (CS 4219). If the screw has reached the level of the plate, it is automatically released from the gripping mechanism of the screw holder (CS 4222).

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The screw should be screwed in further until the locking mechanism has snapped into place over the screw head. A "click" sound will then be heard.

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Removing the fixation pins

The fixation pins (CS 4220) are removed using the inserter for pins (CS 4221).

To insert the fixation pin (CS 4220), push back the sleeve of the inserter for pins (CS 4221). The pin can now be inserted. When the sleeve is released, the pin is firmly locked and can be removed.

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The remaining screws are implanted analogously to the previous steps (pages 15-20).

Important: Visual inspection is then performed to check whether all screws have been locked correctly.

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Use of the bone screw for revision procedures

Bone screws (CS 4203-45-xx) with an external diameter of Ø 4.5 mm are available for revision procedures or whenever necessary.

Implantation is performed analogously to the procedure for self-tapping bone screws (CS 4200-40-xx).

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Optional: Use of the expandable screw

For optimal stabilization and maximum hold of the screw, even in the case of reduced bone quality, the special uNitas™ expandable screw (CS 4202-40-xx) Ø 4.0 mm can be used.

The expandable screw, consisting of a screw and a bolt, can be expanded by screwing in the bolt.

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First, the bolt must be removed from the expandable screw using the screwdriver (CS 4231-02).

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Then the expandable screw (CS 4202-40-xx) is implanted analogously to the procedure for the self-tapping bone screws (CS 4200-40-xx). If necessary, prepare the screw hole using the tap CS 4217.

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The counterpart (CS 4231-01) is positioned on the head of the expandable screw after implantation of the expandable screw. The bolt for the expanding mechanism is inserted through the opening of the instrument. The bolt is then screwed in using the screwdriver (CS 4231-02) and the screw is spread apart.

Important: The counterpart must remain snapped into place in the notches of the expandable screw in order to prevent the screw from simultaneously rotating.

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Use of the bone screw, self-drilling

Self-drilling bone screws (CS 4201-40-xx) Ø 4,0 mm are also available in the uNitas™ system as standard screws.

The self-drilling bone screws do not require any pre-drilling and can be implanted directly.

If desired, the cortical bone can be opened using the awl (CS 4216) and the single (CS 4225) or double drill guide (CS 4226) to facilitate screwing in the screws.

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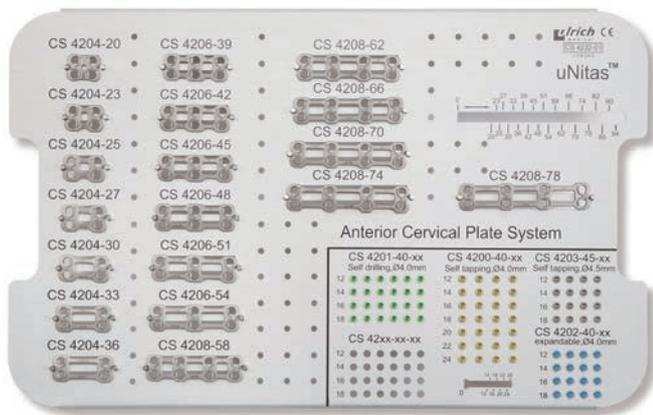
Bone screw revision:

In the case of a revision, the movable arm of the back-out mechanism can be moved backwards to allow the bone screws to be released. To do this, the screwdriver (CS 4219) is guided through the screw removal tool (CS 4228) and positioned in the hexagon socket of the screw.

The fenestration of the screw removal tool (CS 4228) thus shows the direction of the movable arm of the back-out mechanism. The position of the fenestration is also indicated by a laser marking on the T-handle of the screw removal tool (CS 4228).

With mild pressure and a rotational movement of the screw removal tool (CS 4228) in clockwise/ counterclockwise direction (depending on the position of the back-out mechanism), the movable arm of the back-out mechanism is now moved backwards and the screw can be explanted with the screwdriver (CS 4219) at the same time.

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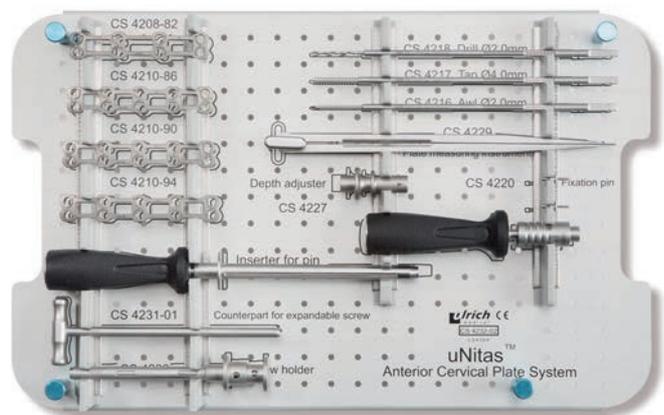
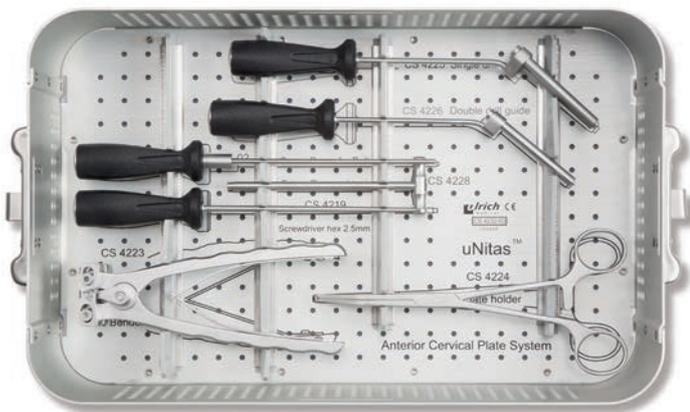
Layer 2 for uNitas plates
and instruments

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Implants	Product number
Bone screw, self tapping , yellow, Ø 4.0 mm, length 12 mm	CS 4200-40-12
Bone screw, self tapping , yellow, Ø 4.0 mm, length 14 mm	CS 4200-40-14
Bone screw, self tapping , yellow, Ø 4.0 mm, length 16 mm	CS 4200-40-16
Bone screw, self tapping , yellow, Ø 4.0 mm, length 18 mm	CS 4200-40-18
Bone screw, self tapping , yellow, Ø 4.0 mm, length 20 mm	CS 4200-40-20
Bone screw, self tapping , yellow, Ø 4.0 mm, length 22 mm	CS 4200-40-22
Bone screw, self tapping , yellow, Ø 4.0 mm, length 24 mm	CS 4200-40-24
Bone screw, self drilling , green, Ø 4.0 mm, length 12 mm	CS 4201-40-12
Bone screw, self drilling , green, Ø 4.0 mm, length 14 mm	CS 4201-40-14
Bone screw, self drilling , green, Ø 4.0 mm, length 16 mm	CS 4201-40-16
Bone screw, self drilling , green, Ø 4.0 mm, length 18 mm	CS 4201-40-18
Expandable screw, incl. bolt , blue, Ø 4.0 mm, length 12 mm	CS 4202-40-12
Expandable screw, incl. bolt , blue, Ø 4.0 mm, length 14 mm	CS 4202-40-14
Expandable screw, incl. bolt , blue, Ø 4.0 mm, length 16 mm	CS 4202-40-16
Expandable screw, incl. bolt , blue, Ø 4.0 mm, length 18 mm	CS 4202-40-18
Bone screw, self tapping , grey, Ø 4.5 mm, length 12 mm	CS 4203-45-12
Bone screw, self tapping , grey, Ø 4.5 mm, length 14 mm	CS 4203-45-14
Bone screw, self tapping , grey, Ø 4.5 mm, length 16 mm	CS 4203-45-16
Bone screw, self tapping , grey, Ø 4.5 mm, length 18 mm	CS 4203-45-18
uNitas plate, 4 holes , width 16 mm, length 20 mm	CS 4204-20
uNitas plate, 4 holes , width 16 mm, length 23 mm	CS 4204-23
uNitas plate, 4 holes , width 16 mm, length 25 mm	CS 4204-25
uNitas plate, 4 holes , width 16 mm, length 27 mm	CS 4204-27
uNitas plate, 4 holes , width 16 mm, length 30 mm	CS 4204-30
uNitas plate, 4 holes , width 16 mm, length 33 mm	CS 4204-33
uNitas plate, 4 holes , width 16 mm, length 36 mm	CS 4204-36
uNitas plate, 6 holes , width 16 mm, length 39 mm	CS 4206-39
uNitas plate, 6 holes , width 16 mm, length 42 mm	CS 4206-42
uNitas plate, 6 holes , width 16 mm, length 45 mm	CS 4206-45
uNitas plate, 6 holes , width 16 mm, length 48 mm	CS 4206-48
uNitas plate, 6 holes , width 16 mm, length 51 mm	CS 4206-51
uNitas plate, 6 holes , width 16 mm, length 54 mm	CS 4206-54
uNitas plate, 8 holes , width 16 mm, length 58 mm	CS 4208-58
uNitas plate, 8 holes , width 16 mm, length 62 mm	CS 4208-62
uNitas plate, 8 holes , width 16 mm, length 66 mm	CS 4208-66
uNitas plate, 8 holes , width 16 mm, length 70 mm	CS 4208-70
uNitas plate, 8 holes , width 16 mm, length 74 mm	CS 4208-74
uNitas plate, 8 holes , width 16 mm, length 78 mm	CS 4208-78
uNitas plate, 8 holes , width 16 mm, length 82 mm	CS 4208-82
uNitas plate, 10 holes , width 16 mm, length 86 mm	CS 4210-86
uNitas plate, 10 holes , width 16 mm, length 90 mm	CS 4210-90
uNitas plate, 10 holes , width 16 mm, length 94 mm	CS 4210-94

Instruments	Product number
AO-Handle	CS 4215
Awl , Ø 2.0 mm	CS 4216
Tap , Ø 4.0 mm	CS 4217
Drill , Ø 2.0 mm	CS 4218
Screwdriver , hex 2.5 mm	CS 4219
Fixation pin	CS 4220
Insert for pin	CS 4221
Screw holder consisting of CS 4222-01; CS 4222-02; CS 4222-03	CS 4222
Sleeve for screw holder	CS 4222-01
Spring for screw holder	CS 4222-02
Clamping element for screw holder	CS 4222-03
Plate bender	CS 4223
Plate holder	CS 4224
Single drill guide	CS 4225
Double drill guide	CS 4226
Depth adjuster	CS 4227
Screw removal tool	CS 4228
Plate measuring instrument	CS 4229
Counterpart for expandable screw	CS 4231-01
Screwdriver for bolt	CS 4231-02
uNitas tray consisting of: CS 4232-01; CS 4232-02; CS 4232-03	CS 4232
Layer 1 for uNitas plates and screws	CS 4232-01
Layer 2 for uNitas plates and instruments	CS 4232-02
Layer 3 for uNitas instruments	CS 4232-03
Lid for uNitas tray	CS 4232-04